

Handwarmer Installation Guide:



1. First you will need to install the power harness to the batteries, to do this you will need to install the red ring terminal (Positive) and the black ring terminal (Negative) onto the positive and negative battery terminals that are located more towards the rear of the chair. You will need to remove the two bolts at the front of the chair in order to tip the seat up to access these terminals. (See photo example 1A)
2. In the second step you will need to install the extension cable, to do this plug one side into the power harness and run the cable out the back side of the chair (See photo example 2A and 2B)
3. In the third step you will need to install the handwarmer/mount onto the joystick, to do this you will need to remove the two screws in the bottom of the joystick. After the two screws are removed you will need to hold the handwarmer mounting bracket in place while you reinstall the longer screws and washers back into those same holes on the bottom of the joystick (See photo example 3A)
4. In this last step all you will need to do is install the other end of the extension cable from step 2 into the open lead coming from the switch that is attached to the handwarmer mounting bracket. (See photo example 4A)
5. Lastly, all you will need to do is secure the extension cable along the same route as the joystick cable using the zip ties supplied in the kit. If your chair has power options like tilt/recline/elevate make sure that the cables are able to move freely with the chair, and make sure that the cables cannot be pinched or pulled on in any way. Slide enclosure over the whole assembly and you are ready to go.

1A)



2B)

**ROUTE EXTENSION CABLE
OUT THE BACK OF THE
CHAIR**



3A)



**LONGER SCREWS AND
WASHERS INSTALLED
WITH HANDWARMER
BRACKET ONTO BOTTOM
OF JOYSTICK**

4A)

