

**POWER**  
**SOCCER**  
**SHOP.COM**



**PS FORCE**  
**SOCCER GUARD**




**Assembly Instructions**

PN 1525

# POWER SOCCER SHOP.COM



2. Use clamps and two 2" bolts to attach each side panel to the wheelchair's leg rest. The single hole and line of 4 holes are for the clamps to set the guard angle.
3. Attach 1 side panel to Right Front Panel. Insert four 1½" bolts in the row of holes in the side panel to attach the guard at the correct height (2 -5 inches from the ground).
4. Attach second side panel to Left Front Panel.

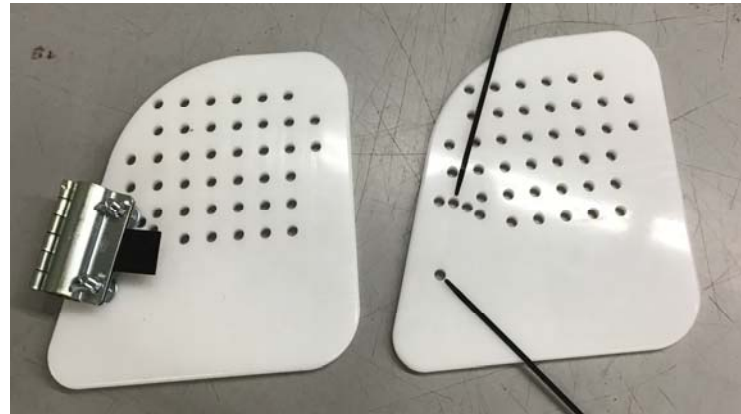

# POWER SOCCER SHOP.COM



5. Select width of guard. Insert four 1½” bolts to attach Left Front Panel to Right Front Panel. Secure with one washer and hex nut for each bolt.



# POWER SOCCER SHOP.COM

