


Quickie QM/ZM 710-310 Guard Mount :

PN 5187

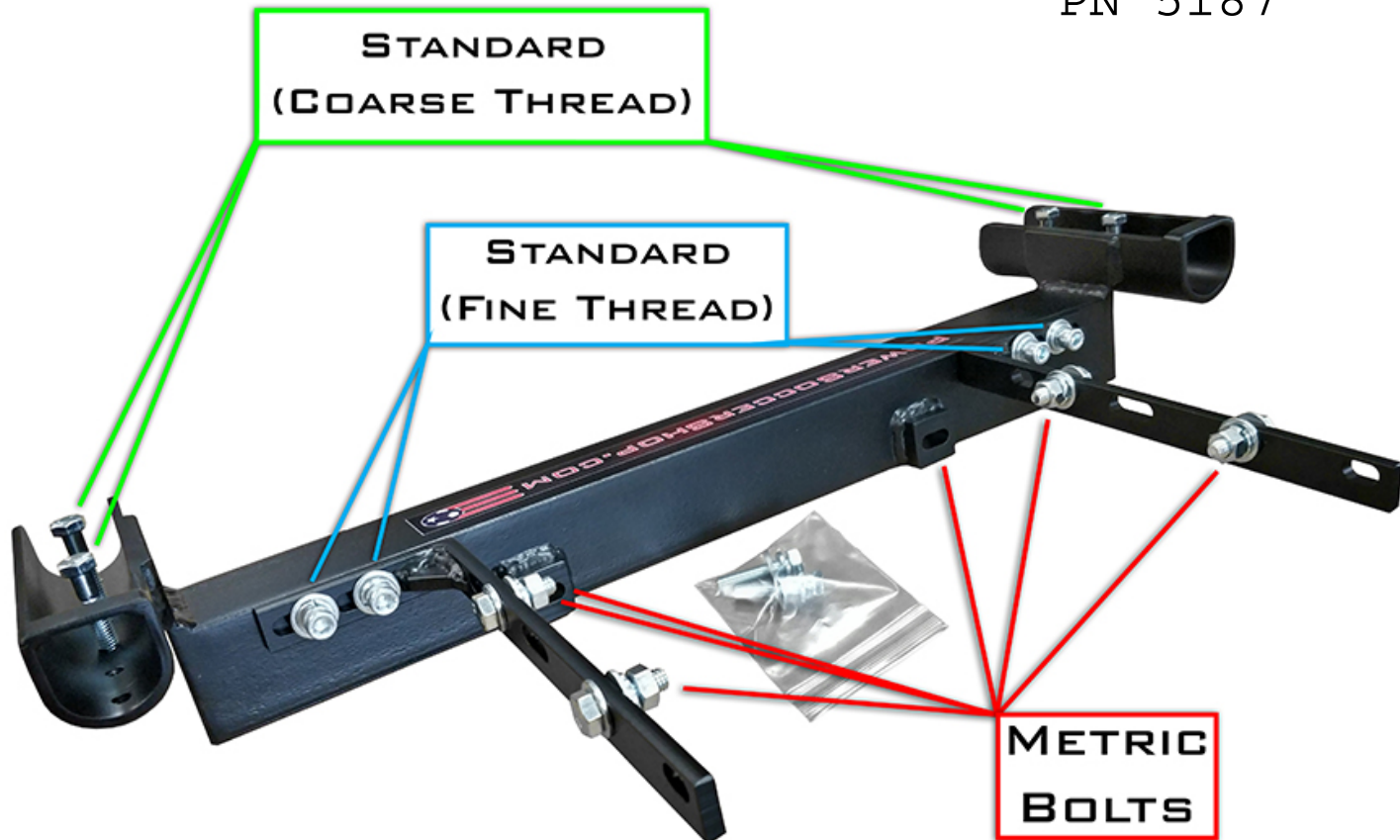


Note: Use reference photo above to determine which bolts to use.

- First you will need to remove the two thumb screws on the front of the chair (if available)
- Next you will need to hold the guard mount in place while you install the two 1½" metric bolts included with the kit, these bolts will be installed in the hole that the thumb screws were taken out of.



**Photos showing thumb screws replaced
with 1½" Metric Bolts.**

- In the next step you will need to install the four 1" metric bolts into the frame on each side of the chair.
- Depending on if the chair has power options on not, you may need to remove and replace existing bolts if they are in the way.

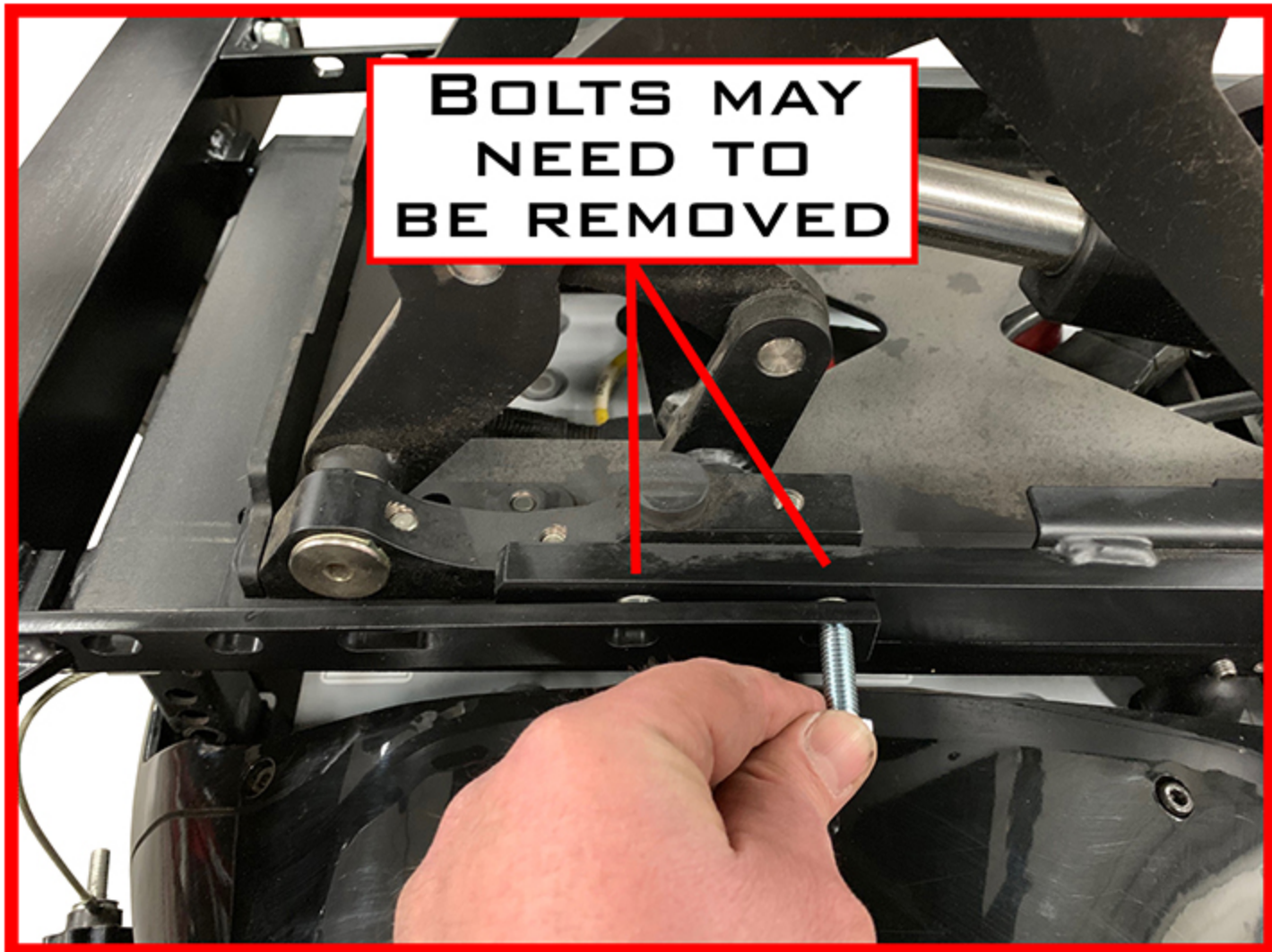

- Lastly, you will need to tighten the four socket head bolts with a ¼" allen wrench to secure the two "L" brackets in place once all of the other bolts are installed.