

ADDING STEEL WEIGHT KIT TO STRIKE FORCE CHAIR

1. BOLT REAR WEIGHT TO CHAIR USING TWO (2) 5/16"-24 X 3/4" SOCKET HEAD CAP BOLTS, FLAT WASHERS AND LOCK WASHERS.



2. REMOVE ANY PARTS THAT MAY BE ATTACHED TO BOTTOM OF PLASTIC SEAT PAN.
3. REMOVE PLASTIC SEAT PAN AND REPLACE WITH STEEL SEAT PAN USING ORIGINAL MOUNTING BOLTS.
4. REATTACH ANY PARTS THAT MAY HAVE BEEN REMOVED FROM THE ORIGINAL SEAT PAN.

