


REMOVING MOTORS FROM STRIKE FORCE CHAIR


1. REMOVE SEAT ASSY. (NOTE: IF CHAIR HAS THE TILT OPTION ,THE TILT ACTUATOR WILL NEED TO BE DISCONNECTED FROM THE SEAT PAN ASSY.)(NOTE: IF CHAIR HAS THE ELEVATE OPTION, THE ELEVATE SUB FRAME WILL NEED TO BE REMOVED)




2. REMOVE BATTERIES

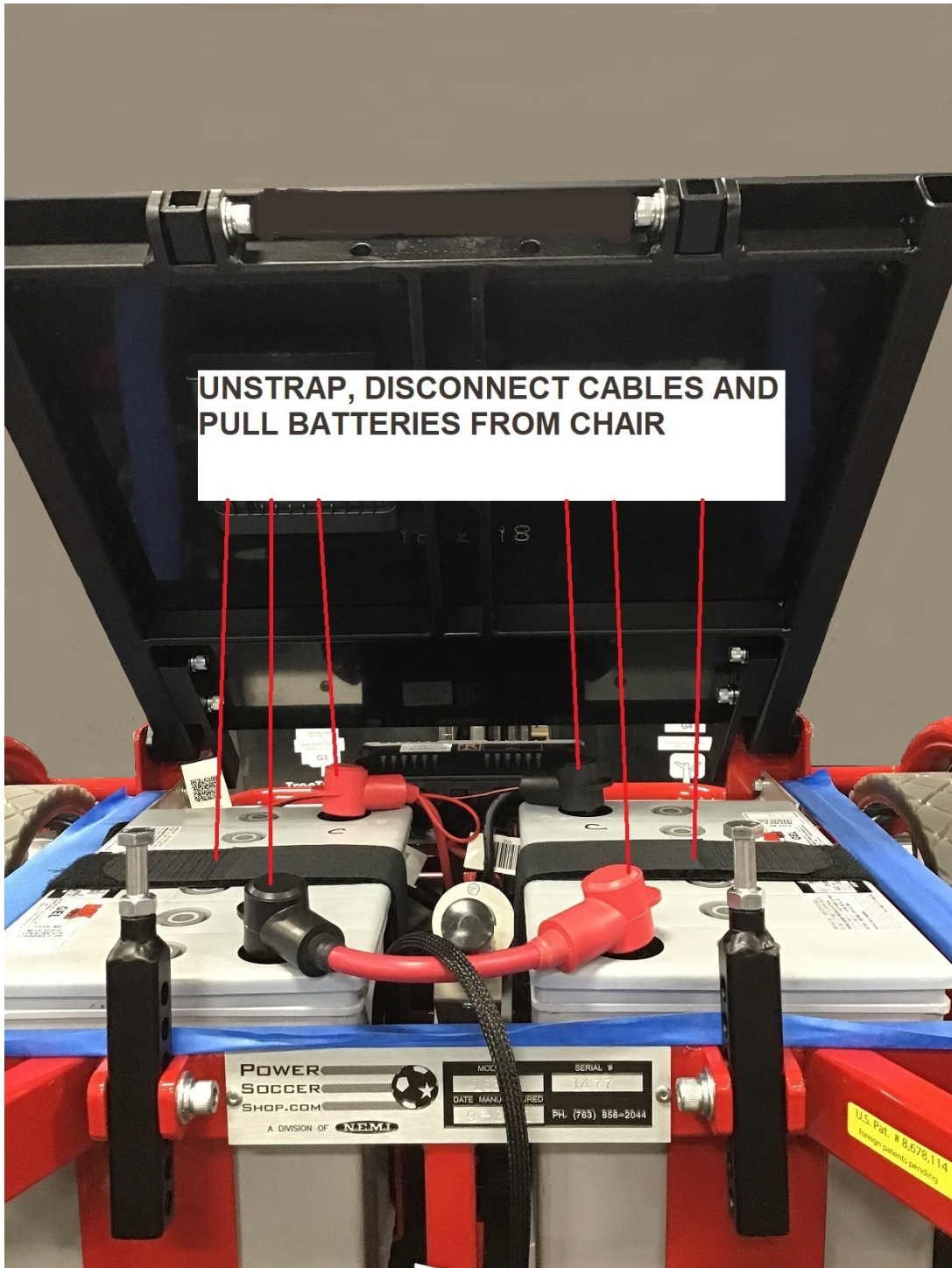


3. REMOVE WHEELS

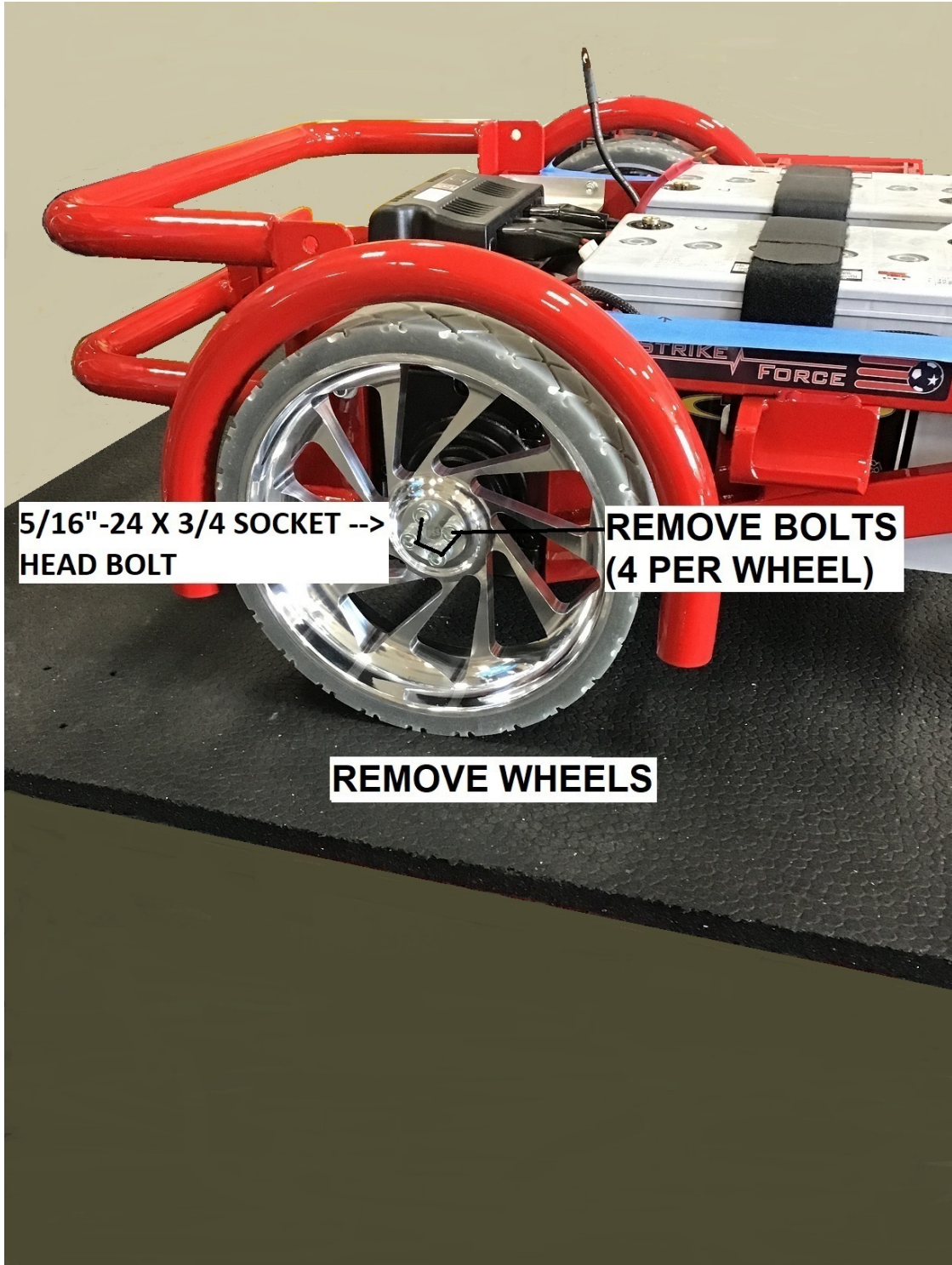

4. REMOVE REAR GUARD

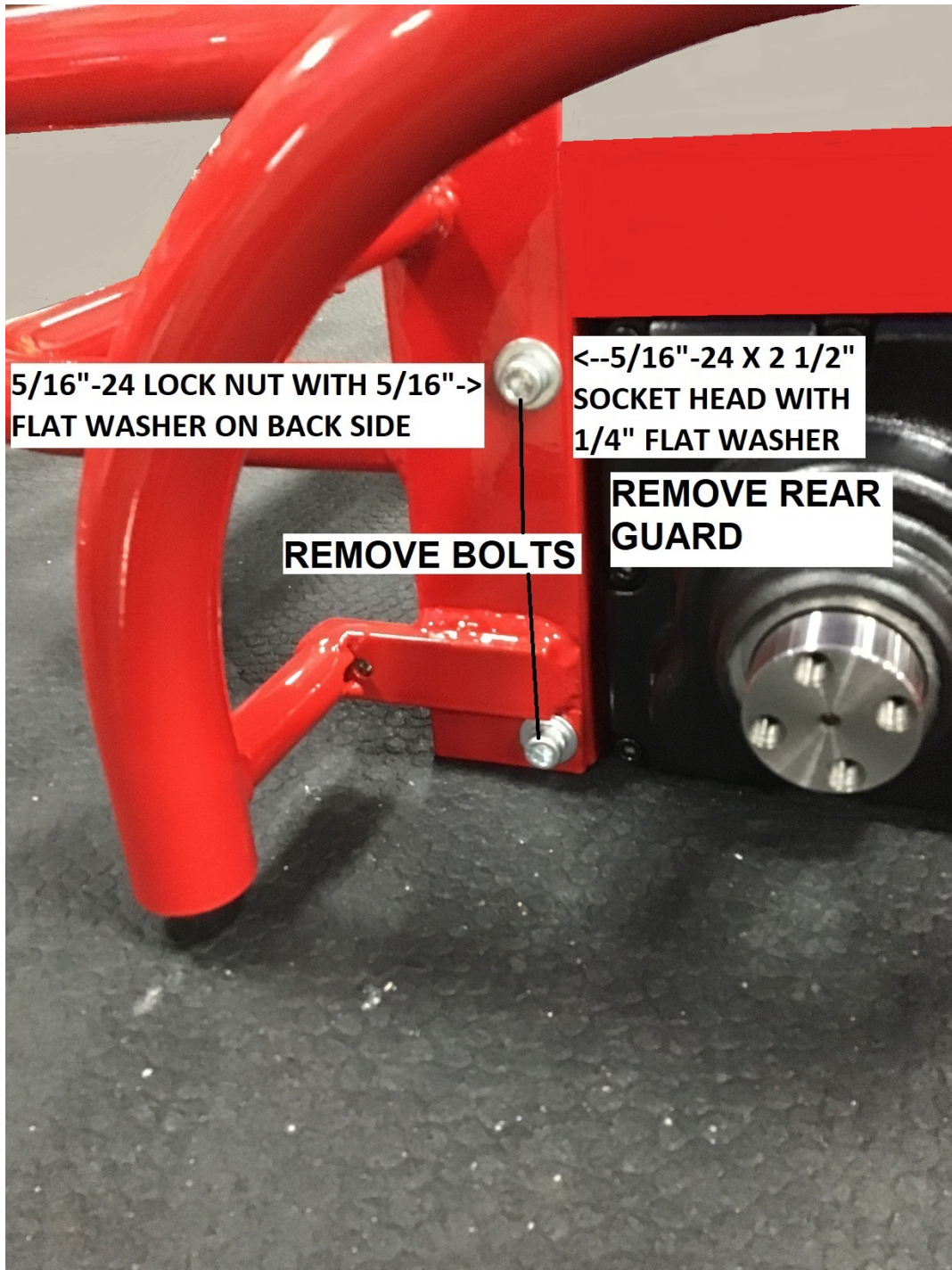

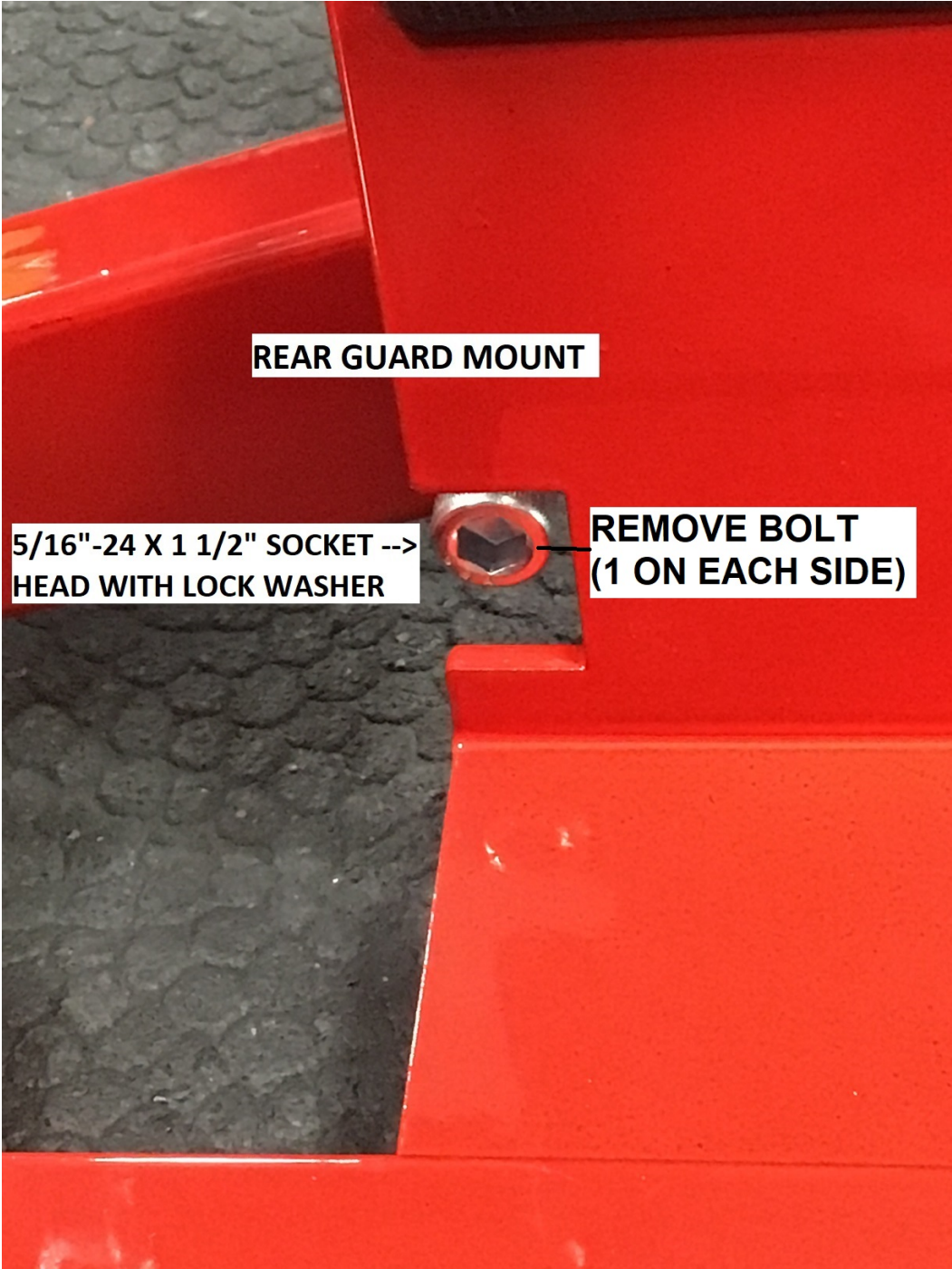



REAR GUARD MOUNT

**5/16"-24 X 1 1/2" SOCKET -->
HEAD WITH LOCK WASHER**

**REMOVE BOLT
(1 ON EACH SIDE)**

5. TIP MAIN FRAME UPSIDE DOWN
6. REMOVE MOTORS

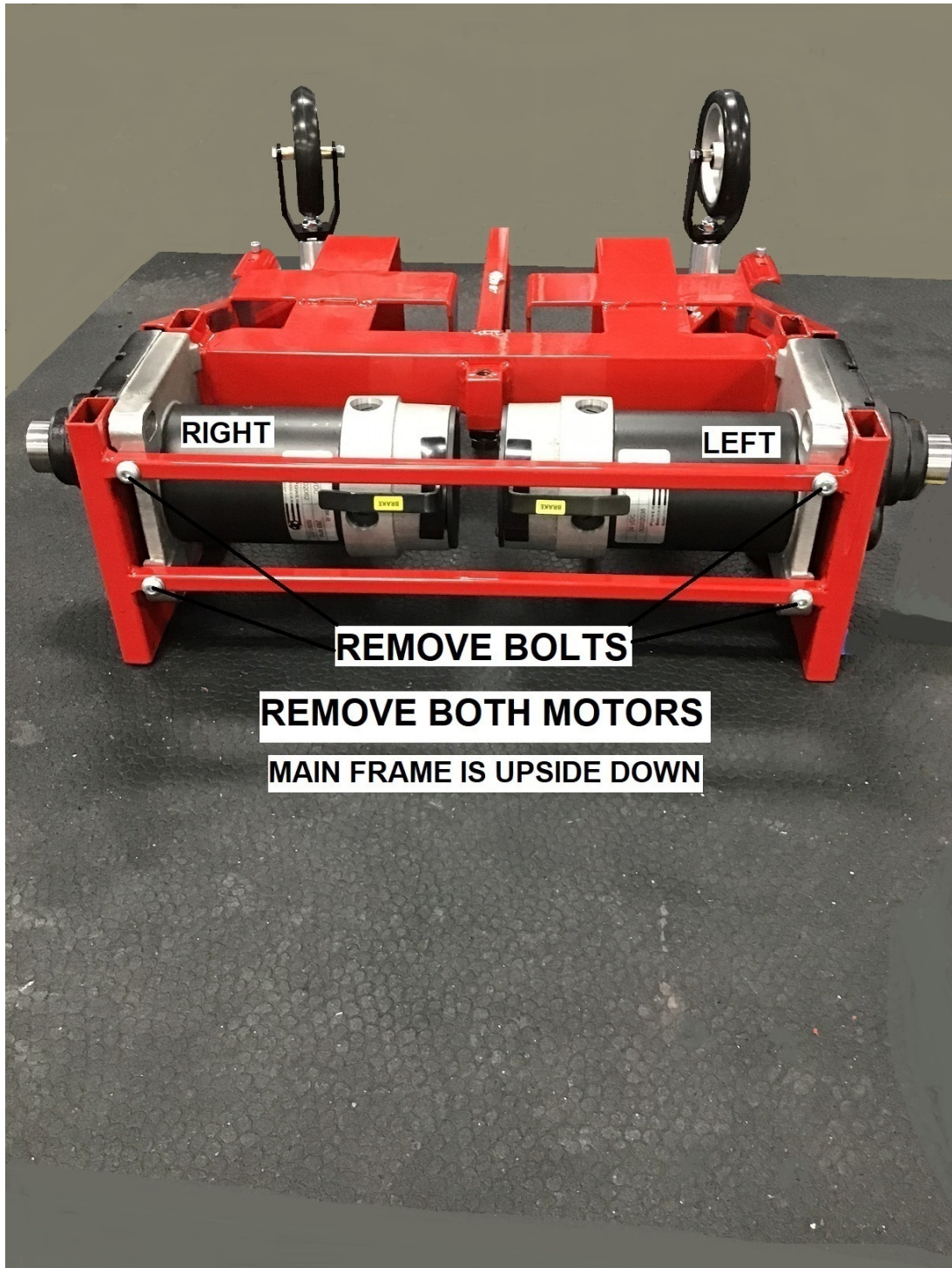

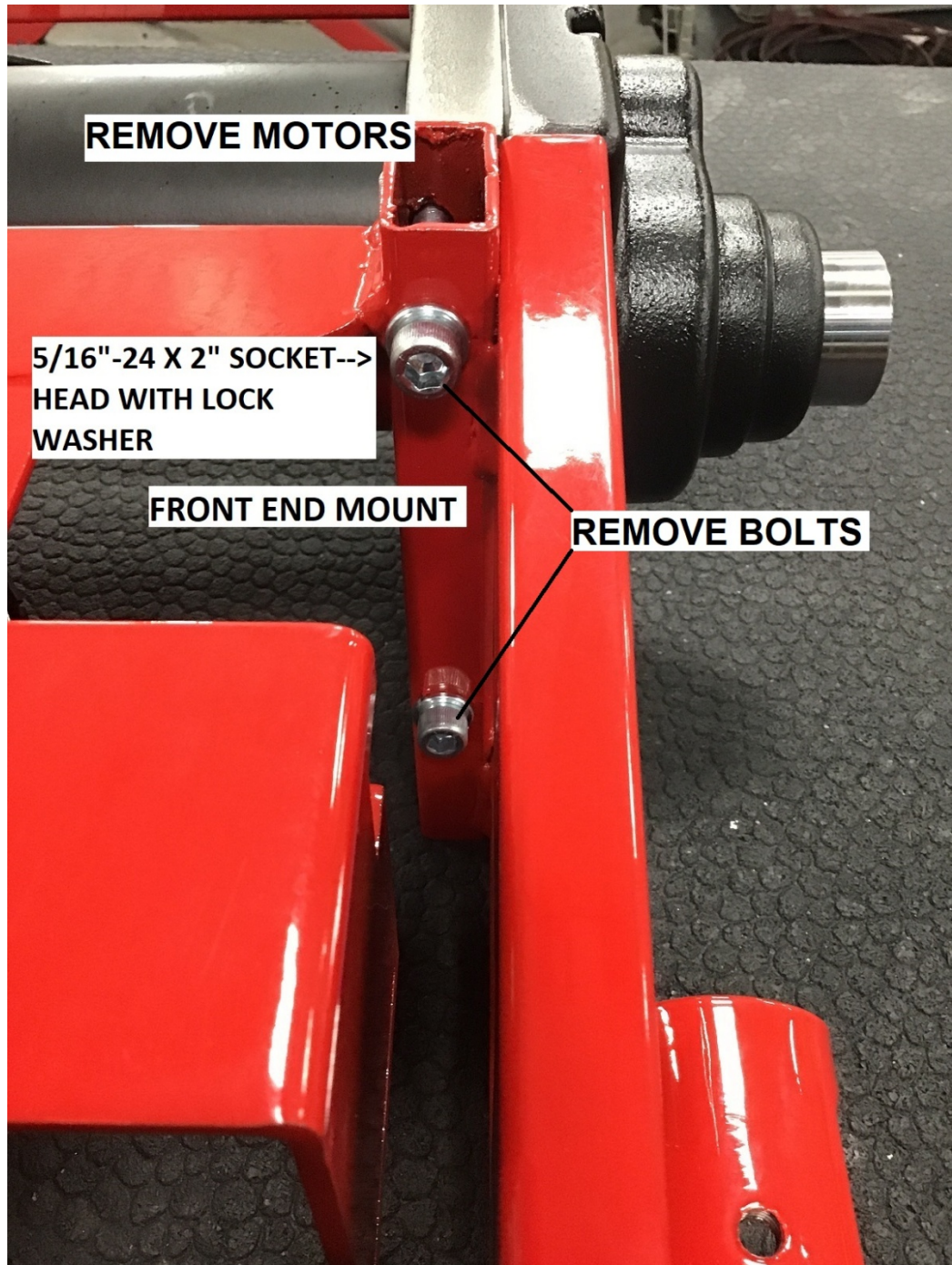

REMOVE MOTORS

**5/16"-24 X 2" SOCKET-->
HEAD WITH LOCK
WASHER**

FRONT END MOUNT

REMOVE BOLTS